

Coro Nipple Shield Fitting Guide

Option 1: Measure your nipple with a ruler:

Measure the diameter of your nipple (not your areola) using the guide below.

- If your nipple diameter measures between 12 mm and 16 mm, the shield should be the perfect fit for you!

We will have Small and Large sizes coming in the future but unfortunately they are not available yet.




Option 2: Size your nipple with our fitting guide

The fitting guide is available below. Please make sure you print at 100% or full scale. The correct size should be a close but comfortable fit around your nipple.

- If the circle feels too small, it may not be the right fit.
- If the circle is much bigger than your nipple, it also may not be the right fit.

We will have Small and Large sizes coming in the future but unfortunately they are not available yet.

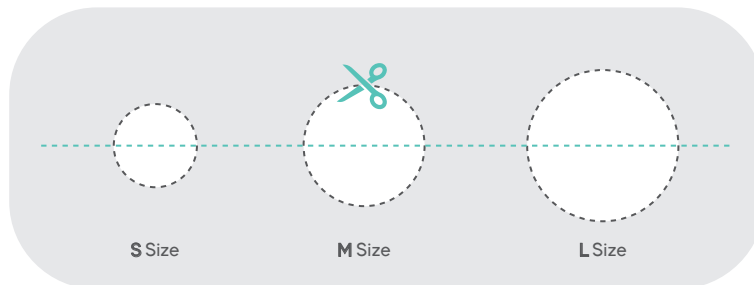

Fold along the line and cut out the dotted line circles.